



**INSTRUCTIONS FOR USE PEP-SET**  
**PURE EFFICIENCY PHACO**


- Use the wrench to screw the ultrasonic tip onto the handpiece. When tightening please ensure that the wrench is mounted precisely on the tip (Careless handling can cause material to transfer from the wrench to the tip. Please check the ultrasonic tip for residues before use).

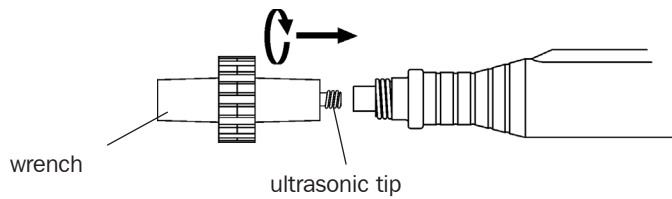

- Turn the infusion sleeve onto the ultrasonic handpiece until it aligns at the base.

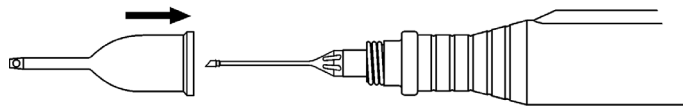


- The infusion sleeve is positioned correctly if it is just beneath the head of the ultrasonic tip. Please pay careful attention not to damage the tube of the sleeve with the ultrasonic tip.

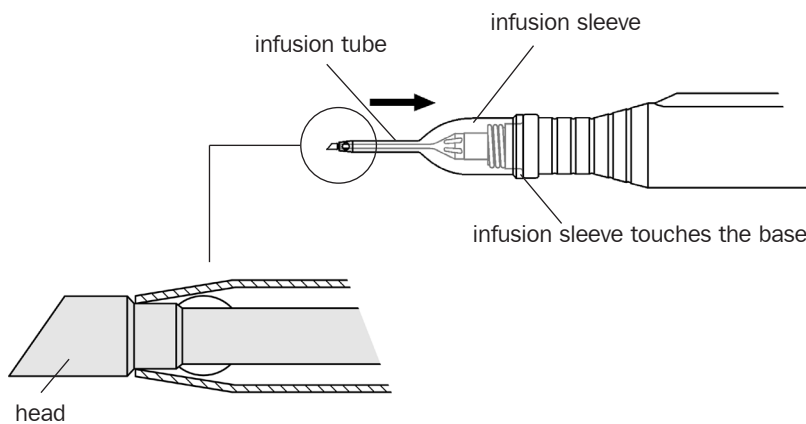

**IMPORTANT:**

**Use a microscope to check the infusion sleeve for correct positioning. Ensure that no residues from the wrench or sleeve are on the ultrasonic tip. Should this be the case, the set must be cleaned or exchanged.**

- Mount the test anterior chamber until it aligns with the base of the infusion sleeve.

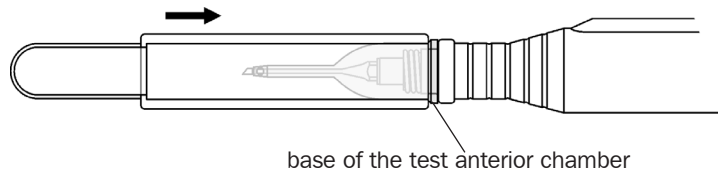

- The PEP set is ready. The handpiece can now be connected to the phaco machine. Please conduct the necessary tests.